

Ets CLAUDE LASSERTEUX S.A

ZONE ARTISANALE – 52800 NOGENT

Téléphone 03 25 31 86 30 – Fax 03 25 31 63 91 – lasserteux.claude@wanadoo.fr

Here are the instructions:

Clean the new instruments before the first sterilisation and after any utilisation.

Decontaminate the instruments respecting the usual methods.

Sterilize the instruments in autoclave at 134°C during 18 minutes.

Our instruments are fabricated to resist intensives cleanings in solution of base of soda.

Be Careful: Don't use any product with chlorine to clean or decontaminate the instruments.

GASPAR Samuel
Responsable Qualité